
MEDPOR Enucleation Implant Inserter

Instructions for Use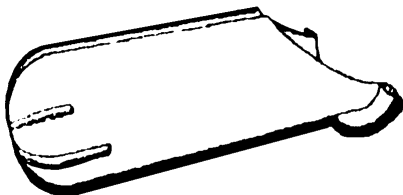

Stryker Howmedica Osteonics

15 Dart Road

Newnan, GA 30265-1017 USA

1-269-324-5346 1-800-962-6558

Fax: 1-877-648-7114


www.stryker.com

English

MEDPOR Enucleation Implant Inserter

Instructions for Use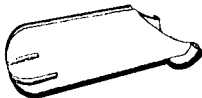

The inserter is provided sterile to aid in MEDPOR Implant placement. Please read the "Instructions for Use" on the reverse.

Use of the Inserter is optional. Surgeons should utilize proper surgical techniques for which they were trained, and their clinical experience to determine appropriate surgical procedures. Successful implantations are technique sensitive. Sound surgical judgement should be used in the selection and the implantation of all MEDPOR Implants.


MEDPOR Enucleation Implant Inserter

– Instructions For Use –

Instructions adapted from notes provided by William White, M.D.



1

After the rectus muscles have been tagged with sutures and disinserted from the globe, the eye is enucleated and hemostasis obtained. Posterior Tenon's capsule is then distracted superiorly and inferiorly with forceps or sutures.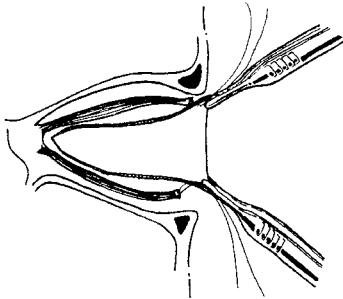



2

The inserter is rolled into a small diameter tube and inserted as deeply as comfortably possible into the intraconal orbit.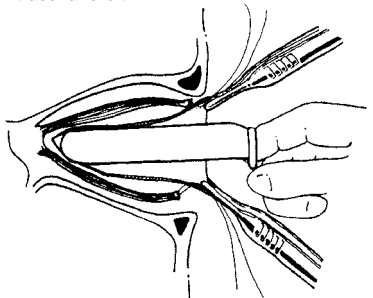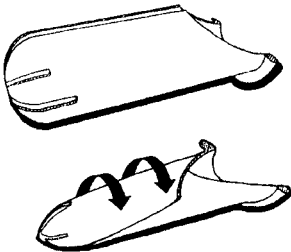



3

The inserter's position deep in the orbit is maintained as the implant is placed in the inserter. The implant is repositioned deeply into the orbit by using a finger or an instrument such as a hemostat or periosteal elevator (not a cotton tipped applicator or other lint or fiber bearing device).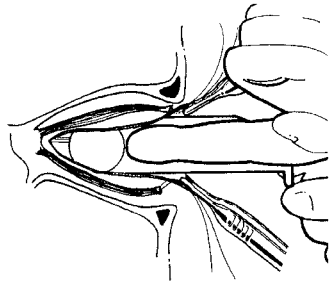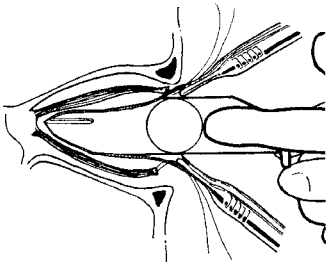

4

Deep implant placement is sustained as the inserter is carefully removed from the orbit. The implant should not advance anteriorly at all as the inserter is withdrawn. The enucleation procedure can then be completed with the implant deep in the orbit and anterior tissues closed under no tension.

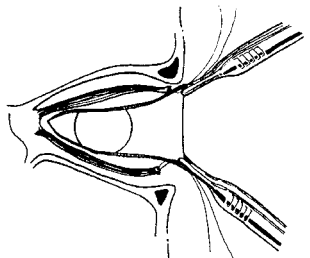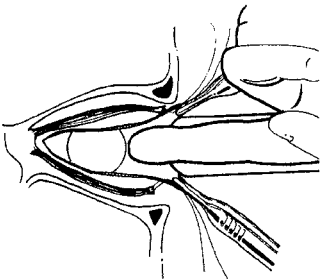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Copyright © 2010, Stryker
Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or services: **MEDPOR** and **Stryker**. All other trademarks are trademarks of their respective owners or holders. ECO-179-01D