


stryker[®]

Natural Iris Conformers

For Immediate Post-Op Enucleation Patients




Available Sizes

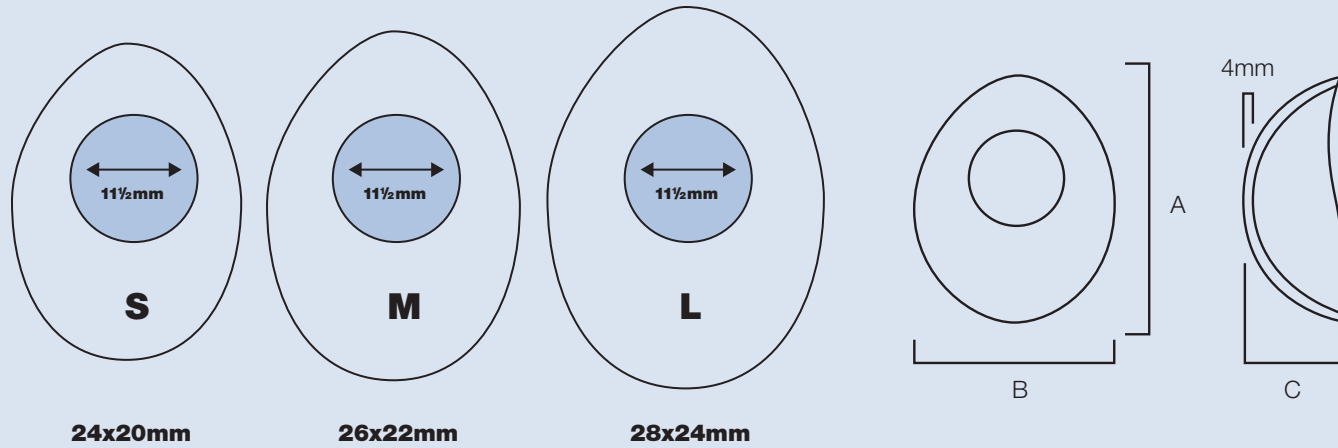

Cat. #	Color #	Size	Color	A	B	C
89120	Color #1	Small	Blue	24mm x 20mm x 9mm		
89121	Color #2	Small	Blue Gray	24mm x 20mm x 9mm		
89122	Color #3	Small	Green Hazel	24mm x 20mm x 9mm		
89123	Color #4	Small	Brown Hazel	24mm x 20mm x 9mm		
89124	Color #5	Small	Med Brown	24mm x 20mm x 9mm		
89125	Color #6	Small	Dark Brown	24mm x 20mm x 9mm		
89126	Color #1	Medium	Blue	26mm x 22mm x 10mm		
89127	Color #2	Medium	Blue Gray	26mm x 22mm x 10mm		
89128	Color #3	Medium	Green Hazel	26mm x 22mm x 10mm		
89129	Color #4	Medium	Brown Hazel	26mm x 22mm x 10mm		
89130	Color #5	Medium	Med Brown	26mm x 22mm x 10mm		
89131	Color #6	Medium	Dark Brown	26mm x 22mm x 10mm		
89132	Color #1	Large	Blue	28mm x 24mm x 10mm		
89133	Color #2	Large	Blue Gray	28mm x 24mm x 10mm		
89134	Color #3	Large	Green Hazel	28mm x 24mm x 10mm		
89135	Color #4	Large	Brown Hazel	28mm x 24mm x 10mm		
89136	Color #5	Large	Med Brown	28mm x 24mm x 10mm		
89137	Color #6	Large	Dark Brown	28mm x 24mm x 10mm		

Natural Iris Conformers

For Immediate Post-Op Enucleation Patients

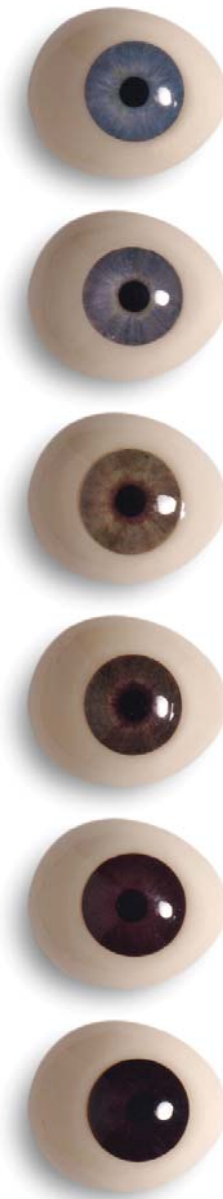

Stryker Craniomaxillofacial
 Kalamazoo, MI 49002 USA
 Telephone: 1-269-324-5346
 1-800-962-6558
 Fax: 1-877-648-7114

Stryker France
 ZAC Satolas Green Pusignan
 Av de Satolas Green
 69881 MEYZIEU Cedex, FRANCE

www.stryker.com

Copyright © 2011 Stryker. Printed in USA.
 Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: **Stryker**. All other trademarks are trademarks of their respective owners or holders.
 Literature Number: xxxx-xxx-xxx Rev. None

Natural Iris Conformers

Natural Iris Conformers are a new rehabilitative conformer that allows the immediate post-operative enucleation patient to heal comfortably without an eye patch. The Natural Iris Conformer covers the socket for the time period required prior to fitting with a custom-made prosthesis.



“The Natural Iris Conformer is an excellent product for immediate post-op enucleation patients. Patients find them to be very comfortable. The biggest difference I have seen is the patient’s ability to get on with social activities before getting their custom prosthesis. I have had many patients that would wear patches or dark glasses. Some would wear their hair over their face or refuse to leave the house until they got their custom prosthesis. With the more normal appearing Natural Iris Conformer, patients feel more at ease.”

John D. Ng, M.D.

Indications for Use

Natural Iris Conformers are temporary devices used after the enucleation of the eye to maintain the shape of the socket and prevent closure or adhesions during the healing process. This product is for temporary use following enucleation and should be replaced by a custom impression-fitted ocular prosthesis fabricated by a trained ocularist.

Natural Iris Conformers are available in six colors: blue, blue-gray, green hazel, brown hazel, medium brown, and dark brown. Each color is available in three sizes: small, medium and large. Approximately 98 percent of patients require a medium conformer.



Comparison

The Natural Iris Conformer is slightly larger than standard clear conformers, but not quite as thick as a custom prosthesis.




“Two of my own children didn’t even notice until I said something.”



“The Natural Iris Conformer presents an immediate emotional and psychological lift at the most critical of times for the patient and their family.”

Doss K. Tannehill, BCO

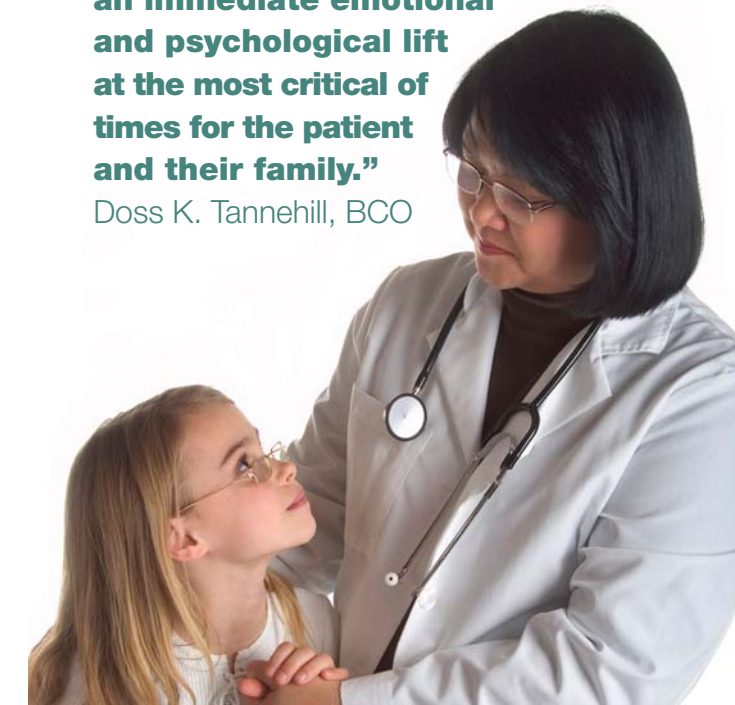

stryker[®]